


# URINE SAMPLE COLLECTION INSTRUCTIONS

## 1 Check kit contents



## 2

**Unscrew lid from urine collection container, place lid with flat surface facing downwards.**

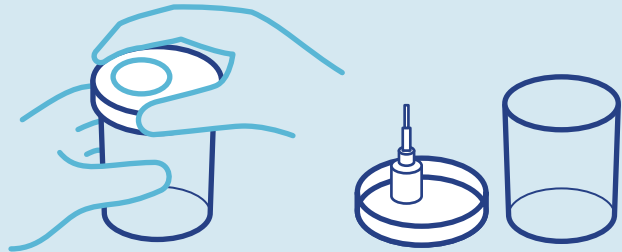

## 3 Collect midstream urine in urine collection container

This means start peeing before collecting. Once about half full, screw lid back on container.



## 4

**Remove safety sticker and insert urine sample tube with lid downwards into lid opening and push.**

The tube will fill up with urine sample.



## 5


**Once filled out, put tube into biohazard envelope and then into reply paid envelope.**

Dispose of container responsibly.



## 6

**Post test kit**

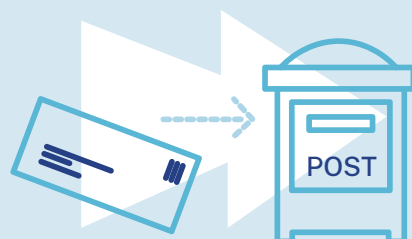

Free postage – no stamp required